**Silhouette**<sup>®</sup>  
 ICONIC EYEWEAR MADE IN AUSTRIA. SINCE 1964.

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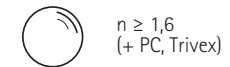
Metal frame made of High-Tech Titanium  
 Lenses: polycarbonate  
 Curving: base 6  
 100 % UV A-, B-, C-protection

Monture en métal en High-Tech Titanium  
 Verres: polycarbonate  
 Courbure: base 6  
 Protection de 100 % de UV A, B, C

Property of Silhouette / © Silhouette

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### Col. / 75-6560

POL GREEN

**metal parts:** ruthenium silky matte  
**outer lenses:** grey with print transparent matte  
**decoration:** black matte  
**temple ends:** black matte

Light Management

### Col. / 75-7010

CLASSIC GREY GRADIENT

**metal parts:** silver silky matte  
**outer lenses:** grey silver mirror with print black matte  
**decoration:** black matte  
**temple ends:** black matte

### Col. / 75-7110

COBALT BLUE

**metal parts:** titanium silky matte  
**outer lenses:** grey silver mirror with print blue matte  
**decoration:** blue matte  
**temple ends:** dark blue matte

POL

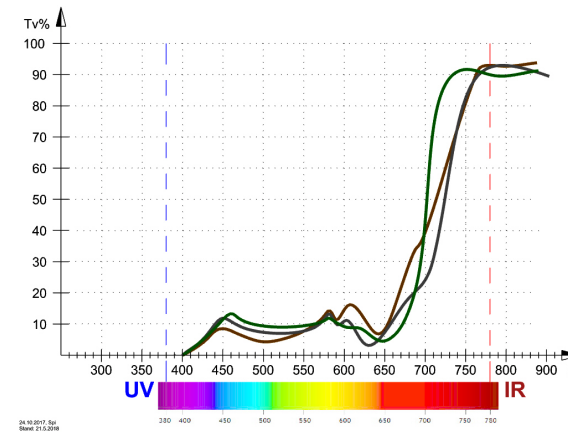
### Col. / 75-7540

POL GREY

**metal parts:** black matte  
**outer lenses:** brown gold mirror with striped print black matte  
**decoration:** black matte  
**temple ends:** black matte

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**3 Classic Must-Have Colours (Brown, Grey, Green)**  
Classical must have colors (Braun, Grau, Grün) / Couleurs classiques incontournables (Brun, Gris, Vert)

## IN-LINE GLAZING

### Milling of optical lenses with nylon-groove

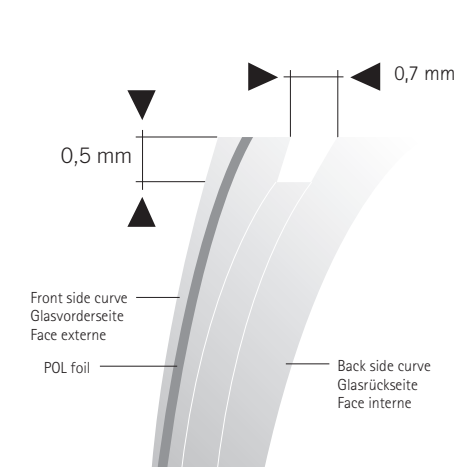
Groove-depth:	0,5 mm
Groove-width:	0,7 mm
Basis curve frame:	5B
Basis curve lens:	4-6 B
Minimum thickness of lens rim:	1,8 - 2,0 mm

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**ATTENTION:** due to minimalist diameters of front frame do not heat under any circumstances!

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Insert lens from behind!  
Note: mounting from the front could crack the lens

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